



Rayovac Stock Number	10ZM
Name	10 Zero Mercury
Cell Chemistry	Zinc Air
Recommended Use	Hearing Aid Instruments
Nominal Voltage	1.45 volts
Designation	ANSI : 7005ZD IEC : PR70 Common : 10
Weight	0.30 grams (.01 oz)
Volume	.08 cc (.005 cu. in.)
Environmental	These cells have no added mercury. Refer to Rayovac Material Safety Data Sheets for more detailed information.
Operating Temperature Range	-10° C to 50° C (14° F to 122° F)
ANSI/IEC Hearing Aid Standard Test	129 hours – 0.7 mA background, 3 mA 100 ms pulse, once every 2 hours, 12 hours/day to 1.05V at 70°F (21° C) and 50% RH.
ANSI/IEC Hearing Aid High Power Test	87 hours – 1 mA background, 5 mA 100 ms pulse, once every 2 hours, 12 hours/day to 1.05V at 70°F (21° C) and 50% RH.
Storage Loss at 70°F (21°C)	Less than 10% in one year with tab applied.
Activation and use	Remove and discard the plastic tab covering air holes. Allow the battery to stand for one minute to insure proper activation. The battery compartment must allow for an adequate oxygen supply to reach air holes.

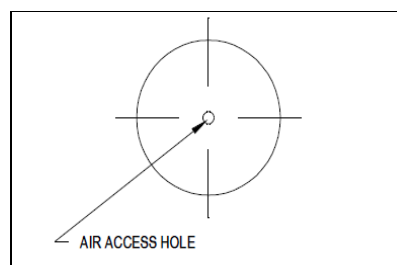

Typical ANSI/IEC Discharge Performance

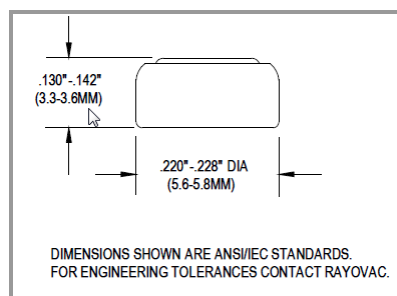